

PET-IR THERMOMETER

Take your Pet's Temperature with ease

PET FRIENDLY FEATURES:

- Device Faster and Cleaner than rectal meter
- Quick Measurement Time (1 sec)
- Mute Setting Option
- Accurate Readings (+/- .2°C)
- Auto-Off Function
- FDA Approved
- Battery Life of 3,000 Measurements
- Cleanable Ear Canal Tip
- Simple Accessible One Measurement Button
- Device Measurement Tip suited for animal use

Source Center, Inc. is the Exclusive Distributor of the *PET-IR Thermometer*

SCI

Source Center, Inc.

Source for Supplies and Inventory Solutions
scisourcecenterinc.com

Source Center, Inc. • PO Box 1387, Mount Laurel, New Jersey 08054 • Office: 856-778-7564 • www.scisourcecenterinc.com

PET-IR Thermometer

Source Center, Inc. is the Exclusive Distributor of the *PET-IR Thermometer*

Quick Guide to first time measuring the Inner Ear Temperature of your Pet

Device Overview

Review Device

- Two AAA Batteries
- Confirm Preferred Units (F° or C°)



Measure (M) Button

Tap and hold Measure Button to put device into *Mute Mode*

Tap the Measure Button until the "House" Icon is shown on screen to measure *Room Temperature*



DIRECTIONS

- Bring the *PET-IR Thermometer* to your pet's inner ear and hold steady.
- Ensure that animal hair is not blocking thermometer tip, as this may cause inaccurate measurements
- Tap and Release bottom button to activate the device
- Tap and Release bottom button a second time, and temperature measurement will occur within one second
- Clean *PET-IR Thermometer* tip with soap and water after each use

PLEASE NOTE

- Temperature measurement greater than 102.5° (F) could indicate a possible fever within dogs and cats, consult your Veterinarian
- Offset Inner Ear Temperature Measurement by plus two degrees (+2° F) from Rectal Temperature Measurement
- For more information and a Normal Rectal Temperature Ranges Chart, please visit: www.scisourcecenterinc.com
- Screen color is not applicable with animals

SCI

Source Center, Inc.

Source for Supplies and Inventory Solutions
scisourcecenterinc.com

SCI

"A Resource Not Just A Vendor!"

Source Center, Inc. • PO Box 1387, Mount Laurel, New Jersey 08054
Office: 856-778-7564 • www.scisourcecenterinc.com

